

G609 Probond Adhesive

Please read this leaflet carefully before you start using the adhesive. Keep this leaflet for future reference.

What is in the pack?

G609 is a water based adhesive.

Each bottle contains 35ml of adhesive.

Distributed by Technovent, Bridgend, CF31 3TB, UK. Tel: 01656 768566

Manufactured by MBI (Wales) Ltd, Bridgend, CF31 3TB, UK. Tel: 01656 768566

Overview.

This water based adhesive is designed for the retention of prostheses to skin for day-to-day use.

User Instructions.

Ensure surfaces to be bonded are clean and free from grease/loose particles (makeup etc).

Apply a thin coating of the adhesive to the back of the prosthesis and allow to dry for 2-3 minutes until adhesive appears clear in colour. The prosthesis can now be adhered to the skin.

Removal - wash off residue from skin and prostheses with soap and water.

Composition/Information on ingredients.

Aqueous dispersion of acrylate copolymer

No hazardous components.

First Aid Measures

Ingestion: Rinse mouth, drink 1-2 glasses of water, do not induce vomiting. Consult a doctor.

Skin Contact: Rinse with running water and soap.

Eye Contact: Rinse immediately with plenty of running water (for 10 minutes). Seek medical attention if necessary.

Inhalation of Vapour: Move to fresh air, consult a doctor if complaint persists.

Storing Your Medicine.

Replace cap securely after use.

Store at room temperature.

Keep out of the sight and reach of children.

Designed for external use only. Avoid use on broken skin.

