



Manufacturer:
TECHNOVENT LIMITED
Unit 5, York Park, Bridgend Industrial Estate
Bridgend, CF31 3TB, UK
Tel: +44 (0) 1656 768566
www.technovent.com



INSTRUCTIONS FOR USE

Probond Adhesive Remover

PRODUCT DESCRIPTION:

Silicone fluid for removing adhesive from prostheses and skin.

INTENDED USE:

Probond Adhesive Remover is designed to be used to remove the residues of the Technovent skin adhesives from external craniofacial prostheses and skin.

INTENDED USERS:

Probond Adhesive Remover is intended to be used by the prosthesis wearer.

INTENDED PATIENT GROUP:

Probond Adhesive Remover is intended to be used on patients who need a maxillofacial prosthesis due to the loss of facial parts due to congenital malformation, tumour surgery or trauma of the craniofacial region.

INDICATIONS OF USE:

Probond Adhesive Remover is useable in most situations – see Intended Use (above) for conditions indicating the use of the device.

CONTRAINDICATIONS/LIMITATIONS OF USE:

Probond Adhesive Remover is suitable to be used with external maxillofacial prostheses on fully healed sites.

If any skin irritation occurs stop using immediately.

PRODUCT RANGE:

Material Code	Description	Fluid
G608	Probond Remover	1cst Polydimethylsiloxane

APPLICATION INSTRUCTIONS:

Carefully peel the prosthesis away from the skin, taking care not to handle the edges. Wipe the skin and surface of the prosthesis with G608 Probond Remover to remove any residue.

STORAGE:

Probond Adhesive Remover is supplied non-sterile, in plastic containers and it is recommended that it is stored in

its original containers. It is recommended that they are stored at ambient temperature (between 15 and 30 degrees Celsius) and out of direct sunlight. Do not use if the packaging is open or has been damaged and do not use after the labelled expiry date.

PRECAUTIONS / RESIDUAL RISKS / SIDE EFFECTS:

-Extremely flammable do not use near naked flames.

-Use in a well-ventilated area.

-This material is designed for external use only on unbroken healed skin.

-In the unlikely event of a localized allergic reaction to the material, stop using immediately and seek advice from Technovent.

- It is essential that all excess adhesive is removed from the silicone daily so as to ensure cleanliness and an accurate fit.

-Extreme care is needed when detaching adhesive retained prostheses especially in regard to the very thin edges that blend into the skin. Tearing of these thin edges will often necessitate the remaking of the prosthesis.

FIRST AID MEASURES:

If swallowed may result in gastric disturbances. Seek medical attention.

Remove excess and wash with soap and water.

May cause temporary irritation and redness. Irrigate with water for 15 minutes, holding eyelid open.

DISPOSAL:

Disposal of the materials should follow local regulations and environmental requirements taking different contamination levels into account.

Designed for external use only. Avoid use on broken skin.

HARMONISED SYMBOLS:



Consult Instructions for Use



Lot Number



Manufacturer



Product Code/Ref



Manufacturing Date



Do Not Store in Direct Sunlight



Expiry Date



Do Not Use if Packaging is Open/Damaged



Store Between 15-30 Degrees Celsius



Flammable