



## Adhesive User Guide



[www.technovent.com](http://www.technovent.com)



+44 (0)1656 768566



[info@technovent.com](mailto:info@technovent.com)



Technovent Ltd



technovent1



Technovent-Ltd

Adhesive retained prostheses are difficult to reposition once adhered.

Therefore it is advisable to place the prosthesis, initially without glue, to check the position and then only use the adhesive when you are sure how the prosthesis will sit.

Before applying adhesive it is important to ensure that both the prosthesis and skin are free from dirt & grease.

We recommend using an alcohol or skin prep wipe for this.

**Adhesives should only be applied to the prosthesis and not directly onto the skin (excluding G626 ProBond Secure Adhesive - this can be applied to the prosthesis and skin).**

### Solvent Based Adhesives



#### How to apply the adhesive:

- Ensure prosthesis is clean and grease-free.
- Apply using the brush supplied.
- Paint a thin layer of adhesive onto the prosthesis.
- Wait 2-3 minutes.
- Apply prosthesis & press firmly onto skin.



#### To remove the adhesive:

For best results use G608 ProBond remover. Remove residue from prosthesis using remover on a textile cloth. The edges of the prosthesis may curl during the removal but they will roll back into position. G608 is also excellent for removal of residue from skin.

### Water Based Adhesives



#### How to apply the adhesive:

- Shake adhesive well before use.
- Using a cotton bud (or similar) apply a thin layer onto the underside of the prosthesis.
- Dry the adhesive until it becomes transparent.



It is advisable to use a hair dryer but keep the dryer at a distance from the prosthesis so no damage is caused. Only when the adhesive is completely transparent can the prosthesis be applied to the skin. Apply the prosthesis and press firmly onto the skin.



#### To remove the adhesive:

Loosen the prosthesis from the skin. (Caution at the edges as these can tear easily) Water based adhesives can be removed from the prosthesis with warm water & soap. Rub gently until all adhesive is removed. Allow to dry.

## Products Available

### Skin Preparation:

**4720** Comfeel Barrier Cream (60ml)

**B-100** Uni-Solve Skin-Prep Wipes (box of 50)

### Solvent Based Adhesives:

**G626** ProBond Secure Adhesive (30ml)

**B-400** Secure Medical Adhesive (30ml)

**B-460** Secure Extra-Strong Adhesive (30ml)

**B-401** Secure Medical Adhesive (57ml)

**B-461** Secure Extra-Strong Adhesive (57ml)

**BT-460** Secure Extra-Strong Adhesive THICK (57ml)

### Adhesive Remover:

**G608** ProBond Remover (100ml, 250ml)

**B-105** Uni-Solve Remover Wipes (box of 50)

### Water Based Adhesives:

**G609** ProBond Adhesive (35ml)

**B-200-R** Daro Regular (57ml)

**B-200-ES** Daro Adhesive Extra Strength (57ml)

**B-230** Daro Adhesive Hydrobond (57ml)

## Skin Preparation



4720  
Comfeel Barrier Cream  
(60ml)



B-100  
Uni-Solve Skin-Prep  
Wipes (box of 50)

## Adhesive Removers



G608  
ProBond Remover  
(100ml, 250ml)



B-105  
Uni-Solve Remover Wipes  
(box of 50)

## Solvent Based Adhesives



G626  
ProBond Secure Adhesive  
(30ml)



B-401  
Secure Medical  
Adhesive  
(57ml)



B-400  
Secure Medical Adhesive  
(30ml)



B-461  
Secure Extra-Strong  
Adhesive (57ml)



B-460  
Secure Extra-Strong  
Adhesive (30ml)



BT-460  
Secure Extra-Strong  
Adhesive  
THICK (57ml)

## Water Based Adhesives



G609  
ProBond Adhesive  
(35ml)



B-200-ES  
Daro Extra Strength  
(57ml)



B-200-R  
Daro Regular  
(57ml)



B-230  
Daro Hydrobond  
(57ml)